COLLANEX

Collagen Particles



Collanex[™] is 100% non-hydrolyzed type 1 bovine native collagen. It protects and retains significantly more native triple helical protein structure, thus allowing superior stability of the molecule and scaffolding through all four phases of wound healing.

Available	Size	HCPCS*	Code

Five Vials A6010

Reorder Number 5898075405



INDICATIONS

of 1 gram each

Collanex[™] Collagen Particles is indicated for use and management of burns, scrapes, sores, ulcers, acute and chronic wounds, superficial, partial and full thickness wounds, infected and non-infected wounds, and minimal exudating wounds. Collanex[™] is easy to apply and is compatible with other topical agents as well as a variety of secondary dressings.

APPLICATION

Cleanse the wound per physician order. Apply Collanex™ Collagen Particles ¼ inch to the entire wound bed surface, including any undermining and or tunneling. Cover the wound with a secondary absorbent dressing to promote thermoregulation and maintain an optimal moist wound healing environment. Frequency of dressing change is based on exudate amount, clinical findings, and per physician recommendations.

CONTRAINDICATIONS

- Known bovine sensitivities.
- Third-degree burns.

APPLICATION

- Single-patient use.
- Sterile until opened.
- Store at room temperature.
- Do not use if package is damaged.

^{*}Products are reimbursable under Medicare in the USA.